

Sterimar Nasal Hygiene Breathe Easy Daily



Category

Cough, Colds and Sore Throat

Manufacturer

CHURCH & DWIGHT UK LTD

Pack size

50ml, 100ml

RRP

£4.49, £6.63

Legal Status

MD, MD



Uses

Cleanses and eliminates impurities (e.g. dust, mucus and allergens). Helps to improve breathing to enhance quality of sleep and helps prevent ear, nose and throat disorders by strengthening the nasal lining's defence. Helps prevent colds by washing out nasal passages.



Format and Ingredients

Nasal spray containing:

100% natural sea water based solution (isotonic solution),



Dosage

Adults and children from 3 years: On first use, prime the spray by briefly pressing the nozzle. For easy breathing and to moisturise the nasal lining: Keeping the head straight, gently put the nozzle in the nostril. Spray once or twice in each nostril. Let any excess solution flow out and then blow the nose. For a deeper cleansing of the nasal cavities: Keeping the head tilted to one side, gently put the nozzle in the upper nostril. Dispense a prolonged spray. Let any excess solution flow out of the other nostril and then blow the nose. After each use, clean the nozzle in hot water and dry. Use 2-6 times a day for as long as needed or as long as recommended. Suitable for pregnant and breastfeeding women.